


— Soft laser expertise

premio 30

Laser duo

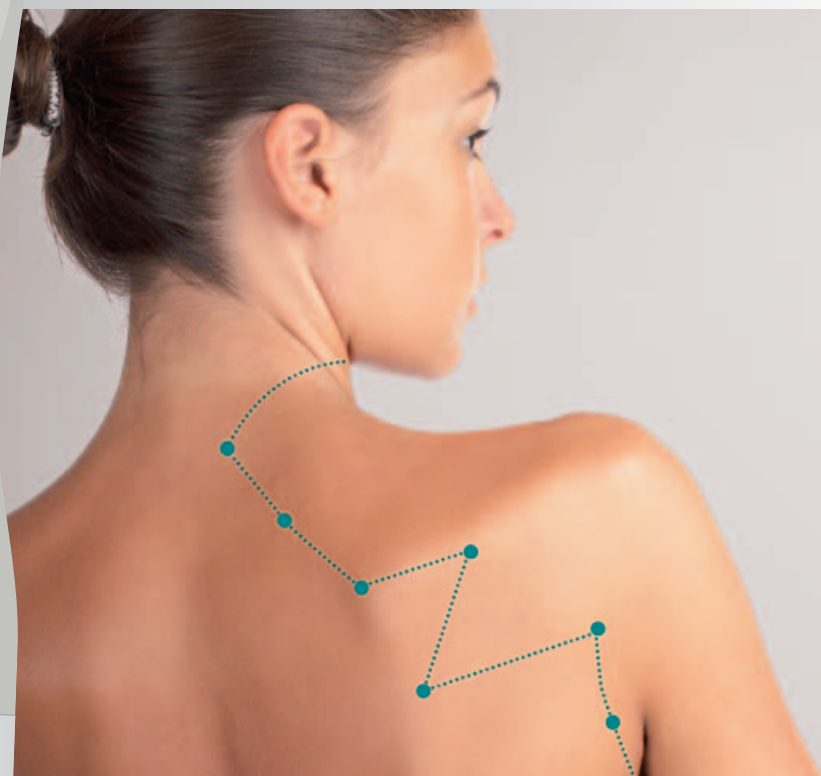

Acupuncture
Auriculotherapy
Addictions treatment
Reflexology

Sedatelec




premio 30

Laser duo

Painless stimulation of body and auricular points



**4 treatment
modes**

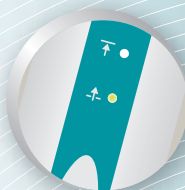

**2 powers,
adaptable for
superficial or
deeper points**



**10 hours
working time**
Rechargeable
battery

Between tradition and modernity: **safe and effective**

Acupuncture

Tonification

Controlled, rapid tonification of acupuncture points.



Dispersion

In a few seconds, the Qi will be dispersed and you will remove energy stasis.



Harmonisation

This treatment rebalances the point and the circulation in the meridian by alternating impulse rates.



Auriculotherapy

Frequency stimulation

This selection will guarantee stimulation of « pathological » auriculotherapy points (pain, psychological or functional problems, addictions).



Innovation and effectiveness of frequencies

New impulse emission dynamics


Wobulation (100-10 000Hz)

Exponential acceleration

Tonification occurs as a result of an **acceleration** in **impulse emission rate**, which has an energising effect.



Tonification
impulse rate

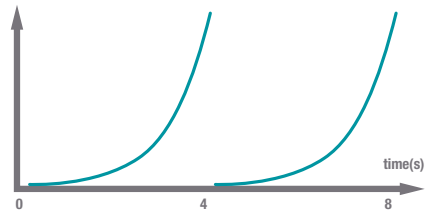


Exponential deceleration

Dispersion is obtained by **slowing** the **impulse emission rate**.



Dispersion
impulse rate

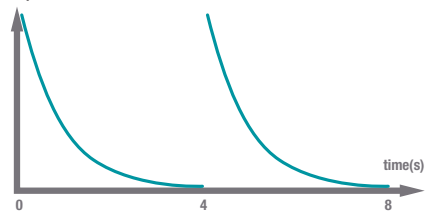


Alternating acceleration and deceleration

Alternating **acceleration and deceleration** of the emission rate leads to harmonisation of the point.



Harmonisation
impulse rate

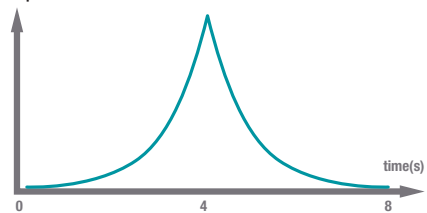

Scanning

According to the 7 Nogier frequencies

The message is contained in the frequency for this auriculotherapy programme.


It provides a successive emission by **scanning all the Nogier frequencies** (frequencies A, B, C, D, E, F, and G)




Renowned experience in Pulsed Laser

Wavelength

The 905nm infra-red emission passes through the skin barrier and reaches the deep cells, entering entirely safely without being absorbed by skin water, haemoglobin or melanin.

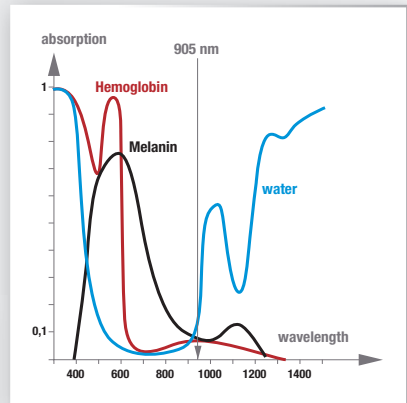

Peak value

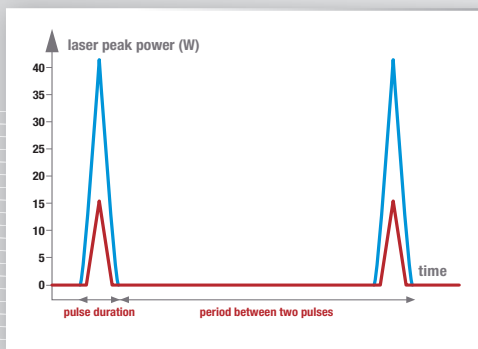

The laser beam is emitted as short impulses of high peak power. This method of emission gives the laser beam the **necessary power to enter tissues** entirely safely with no destructive heat effect, carrying the useful information to the site point.

Adjustable power

15 or 40 Watts power is adjustable depending on the desired depths

- **Superficial points:** surface points (face, ears)
- **Deep points:** other body points

Application time

Short treatment of 30 seconds for each point. This application time provides the optimal energy per point taking account of the physiological body reaction time.

Sedatelec +:
frequencies wobulation for acupuncture

The Sedatelec signature:
Nogier frequencies for auriculotherapy



premio 30

Laser duo



**Effective, light
and ergonomic**

**A wireless laser,
ideal for the
consulting room
and on the move**



*Invisible laser emission. Do not point the laser beam towards eyes. 3B classification according to IC60825-1
Caution : Federal law restricts this device to sale by or on the order of a physician.*

Ask for the "further information" documentation on the Premio 30 from your distributor or by contacting sedatelec@sedatelec.com

Your distributor



Made in France

Sedatelec • Chemin des Mûriers • F-69540 Irigny • Lyon • France • Tél. +33 (0) 472 663 322
Fax + 33 (0) 478 508 903 • sedatelec@sedatelec.com • www.sedatelec.com

