


Frequency Electrical Stimulation by Sedatelec

AGISTIM DUO

99 Hz A-G



99 Hz A-G

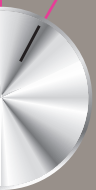

ACUPUNCTURE
PAIN RELIEF
AURICULOTHERAPY
AESTHETIC MEDICINE

**Sedatelec**

AGISTIM DUO

Professional, multi-functional by design



Variable
Frequencies from
1-100 Hz



Timer
treatment modes



4 channels,
up to 16 needles

To enhance the effects of acupuncture treatments, effectively relieve pain or reduce localised unsightly lipodystrophies, **AGISTIM DUO** is your immediate choice.



Electro-Acupuncture

more efficient, more immediate !

Electrical stimulation increases the action of needles. It can even replace their manipulation .

— **Tonification:** use a low frequency (2 to 5Hz) and long stimulation time.

— **Dispersion:** use a high frequency (70-80Hz) and short stimulation time.

The electrical independence of the channels enables you to have several needles differently stimulated at the same time, easily choosing their emission mode.

The adhesive skin electrodes make it easy to treat equally children or fearful patients;

Pain, chronic symptomatic problems, addictions all respond particularly well to this electro-acupuncture



Auriculotherapy

Treatment of Auriculotherapy points with **Nogier frequencies**. The Styl'Module Pen offers precision and working comfort.

It can be used in continuous, discontinuous or scanning mode.

— The 7 frequencies available are the reference Nogier frequencies

A (2,28 Hz), **B** (4,56 Hz), **C** (9,12 Hz), **D** (18,25 Hz), **E** (36,5 Hz), **F** (73 Hz), **G** (1,14 Hz).

The 1.14 Hz frequency is also called the Universal Frequency (U).

Symptomatic problems, musculo-skeletal problems, pain, psychological problems, ... all indications for Auriculotherapy

Pain relief treatments scientifically confirmed

Pulsed electrical stimulation is now a proven technique validated by many scientific studies.

— Gate control mobilisation of nerve fibres


to block pain before it reaches the central nervous centres.

Loco dolenti, by segmental or supra-segmental control, at a 10 to 80Hz frequency

— Release of endorphins, opioids

which control pain sensation, through low frequency stimulation from 1 to 10Hz.

Through needles or adhesive skin electrodes.



**Musculo-skeletal disorders,
nociceptive pain,
neuralgia, phantom pain,
acupuncture, analgesia,...**

Aesthetics & ElectroLipolysis


a gentle alternative
for visible results

Frequency-controlled stimulation by the Agistim Duo produces excellent results in reducing localised lipodystrophies in a few weekly sessions.

Stimulating the dermis myofibrils returns tone to the skin reducing lines and weak points.

The Lipolysis kit make it possible to connect up to 16 needles simultaneously.

Sedatelec ASE specific needles are specially designed for this application (diamond tipped).



**These aesthetic results are
enhanced by complementary
acupuncture treatment
particularly targeted to reduce
appetite and improve water
distribution in the body**

Efficacy through SCFO* innovation

An ingenious program of permanent impulse frequency fluctuation around the chosen frequency greatly reduces the risk of physiological habituation to stimulation.

**Statistically Controlled Frequency Oscillation: a Sedatelec exclusively patented process*

Simplicity

- 2 independent stimulators in the same instrument (Duo function)
- Individual keys to set time, frequency and intensity
- Multi-lingual control screen (EN, FR, ES, PT, DE, IT, NL)
- 4 independent channels
- 2 to 16 colour connectors
- Stimulation through **needles** and adhesive skin **electrodes**
- Timer, up to 60 minutes

Maximum safety

- Total insulation from the main supply
- Safety function prevents treatment beginning without resetting the intensity to zero.
- Stop alert if a frequency parameter or emission mode is changed during treatment
- 4 electrically insulated channels



Thanks to their original design, the needle connectors ensure perfect contact with the needle, with at the same time guarantee of complete insulation of the skin.

During treatment, the patient can adjust the intensity of each channel by himself as no incorrect manoeuvres are possible.

The Sedatelec signature: **Nogier frequencies**

The 7 pre-programmed Nogier frequencies (from A to G) can be selected individually or in scanning mode for auriculotherapy or acupuncture stimulation.



AGISTIM DUO

Excellence in its field

Multiple functions, design, simplicity, safety and efficiency are the features of the AGISTIM DUO which you and your patients will appreciate on a daily basis.

This confirms your professional approach and the quality of your treatment.



Ask for the «For more information» documentation about AGISTIM DUO from your supplier or by writing to sedatelec@sedatelec.com

LAGISTIM DUO is not indicated for use in myostimulation rehabilitation. No FDA approved (USA).

Your distributor



Sedatelec • Chemin des Mûriers • F-69540 Igny • Lyon • France • Tél. +33 (0) 472 663 322
Fax + 33 (0) 478 508 903 • sedatelec@sedatelec.com • www.sedatelec.com

1 - 9,9 Hz

10 -

1 - 9,9 Hz

10 -