

premio 20 DT  
electrical detection and stimulation



AURICULOTHERAPY

ACUPUNCTURE


REFLEXOLOGY





## 40 years of innovation and services dedicated to Auriculotherapy

Created by **Dr. Paul Nogier**  
in the 1950s, auriculotherapy  
has been recognised  
since the 1990s by the WHO.  
This technique is now practiced  
throughout the world.

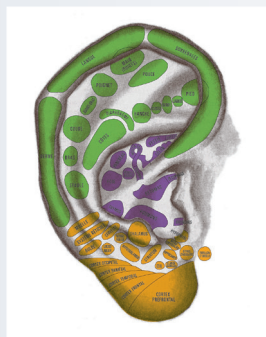

**The ASP® Original** or Semi-Permanent Needle  
has been created and manufactured in France by  
Sedatelec in collaboration with Dr. Nogier.

From R&D, through design  
and development, to manufacture  
everything is **“Made in France”**.


Sedatelec guarantees its products for 2 years  
and provides after-sales services from its  
premises in Lyon for up to 10 years after sales  
have been discontinued.

ISO 13485  
CE0123



# premio 20 DT

electrical detection and stimulation

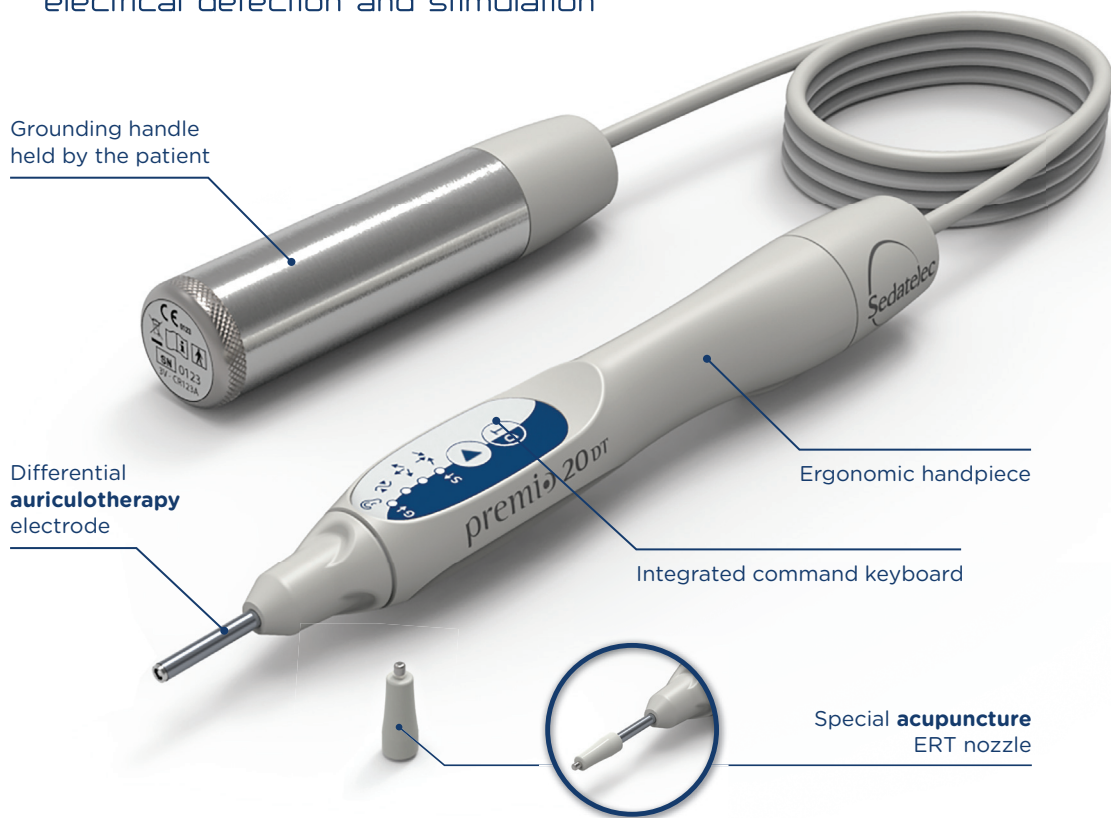

**Sedatelec** was the original developer of the first auricular point detectors (Diascope®, Servoscope®, Agiscop®) and has concentrated all of its skills and expertise in the production of a new generation, with the **Premio 20**.

## Revolutionary design

- PRACTICAL:** no cable or accessory part to install
- ERGONOMIC:** the first device with direct reading from the ear
- ACCURATE:** 9 level differential detection with LED display
- INNOVATIVE:** frequency wobulation in stimulation mode between 1 and 100 Hz
- SIMPLE:** audible trigger threshold adjustment
- COMFORTABLE:** automatic adaptation of stimulation intensity




## Detection

- Sufficiently low measurement current not to interfere with the point.
- Constant measurement electrode pressure on the epidermis.
- **Highly accurate** differential detection system (point and environment).
- Low (Gold) or high (Silver) impedance detection possible.
- **Adjustable audible trigger threshold.**
- Continuous measurement display on **9 levels.**
- A grounding handle held by the patient which isolates the practitioner from the measurement circuit.

## Differential detection

### Measurement of the point environment

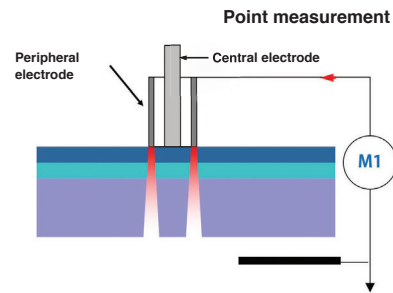

## 3 levels of led light indication



Slow flashing



Rapid Flashing




Fixed light

Automatic setting of the audible trigger threshold through measurement of the «0» point

D VERSION

DT VERSION

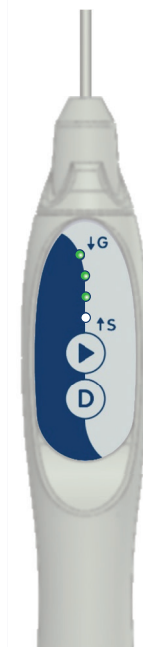



## Stimulation

- **Without taking your eyes** off the point detected, simply pressing on the «D/T» button triggers stimulation by scanning the 7 Nogier\* frequencies (1.14 Hz included).
- **Automatic adaptation** of stimulation intensity: no pain or burning sensation.
- Emission of a soft and fast «BLEEP» indicating to both patient and therapist that the stimulation is ongoing.
- Duration time set by a louder «BLEEP» every 30 seconds: average time required to incapacitate the point.
- Pressing again on «D/T» restarts detection without taking your eyes off the ear which is being investigated.

\*The Nogier frequencies are not recognised by Health Canada.



auriculotherapy




## Detection

- **Downward measurement** of the impedance point.
- Special «Body» **ERT nozzle** with soft tip for perfect sliding.
- Progressive diode lighting proportional to the measurement.
- Direct instantaneous reading of the measurement on the stylet confirmed by an «Audible BLEEP».
- **Memorisation of the skin point** detected with the lowest impedance.
- **Reliability:** grounding handle held by the patient isolates the practitioner from the measurement circuit.




# Stimulation

- Without taking your eyes off the point detected, simply pressing on the «D/T» button triggers appropriate simulation for the patient's needs.
- **INNOVATION:** emission by frequency wobulation:
  - Alternate for **harmonisation**
  - Decreasing from 100 to 1 Hz in **dispersion**.
  - Rising between 1 and 100 Hz in **tonification**
- Option of working with Nogier\* frequency scanning.
- Automatic adaptation of stimulation intensity: no pain or burning sensation.
- Emission of a regular «BLEEP» indicating to the patient and to the therapist that stimulation is ongoing.
- Duration time set by a louder «BLEEP» every 30 seconds.
- Pressing again on «D/T» restarts detection without taking your eyes off the ear which is being investigated.

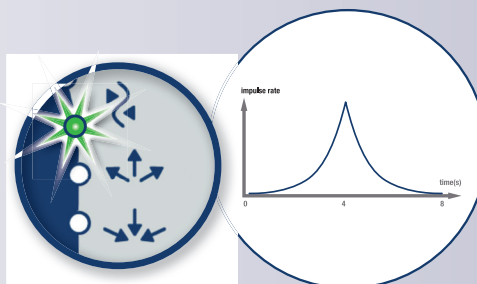


## HARMONISATION

This stimulation re-equilibrates the point and circulation in the meridian by alternating accelerations and decelerations of impulse frequency.

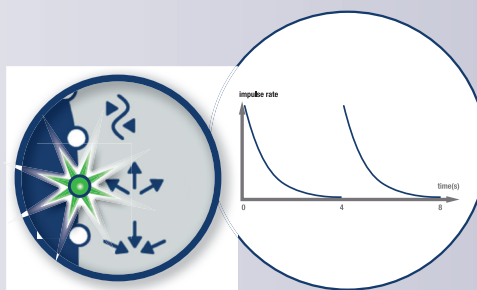

## DISPERSION (100 to 1 Hz)

In a few seconds you will disperse the Qi and remove energy stasis. Dispersion is obtained by decreasing the impulse emission frequency.

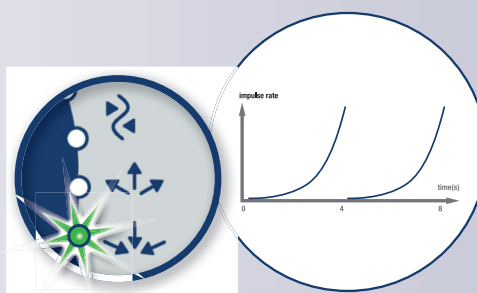

## TONIFICATION (1 to 100 Hz)

Rapid controlled stimulation of acupuncture points. Energising effect by accelerating impulse emission frequency.

«BLEEP» every 30 seconds

## Effective, light and ergonomic

An ideal  
**detector/stimulator**  
for the consulting  
room and for  
portable use.



Distributor stamp

CE 0123

Manufactured in France



Chemin des Mûriers • F-69540 Irigny (Lyon) • France  
Tel: +33 (0) 472 663 322 / Fax: +33 (0) 478 508 903  
contact@sedatelec.com • [www.sedatelec.com](http://www.sedatelec.com)

Before use, read the instructions carefully.



SED-PREMIO20-EN-2020-04 • novarte.creation@wanadoo.fr • 04 78 28 55 44 • photos : istock • © Sedatelec