

premi^o 20 DT

electrical detection and stimulation



AURICULOTHERAPY

ACUPUNCTURE

REFLEXOLOGY

Sedatelec[®]
YOUR WAY OF CARE,
OUR QUALITY DEVICES



40 years of innovation and services dedicated to Auriculotherapy

Created by **Dr. Paul Nogier**
in the 1950s, auriculotherapy
has been recognised
since the 1990s by the WHO.
This technique is now practiced
throughout the world.



The ASP® Original or Semi-Permanent Needle
has been created and manufactured in France by
Sedatelec in collaboration with Dr. Nogier.

From R&D, through design
and development, to manufacture
everything is **“Made in France”**.


Sedatelec guarantees its products for 2 years
and provides after-sales services from its
premises in Lyon for up to 10 years after sales
have been discontinued.

ISO 13485
CE0123

CONÇU & FABRIQUÉ
EN FRANCE

premio 20^{DT}

electrical detection and stimulation

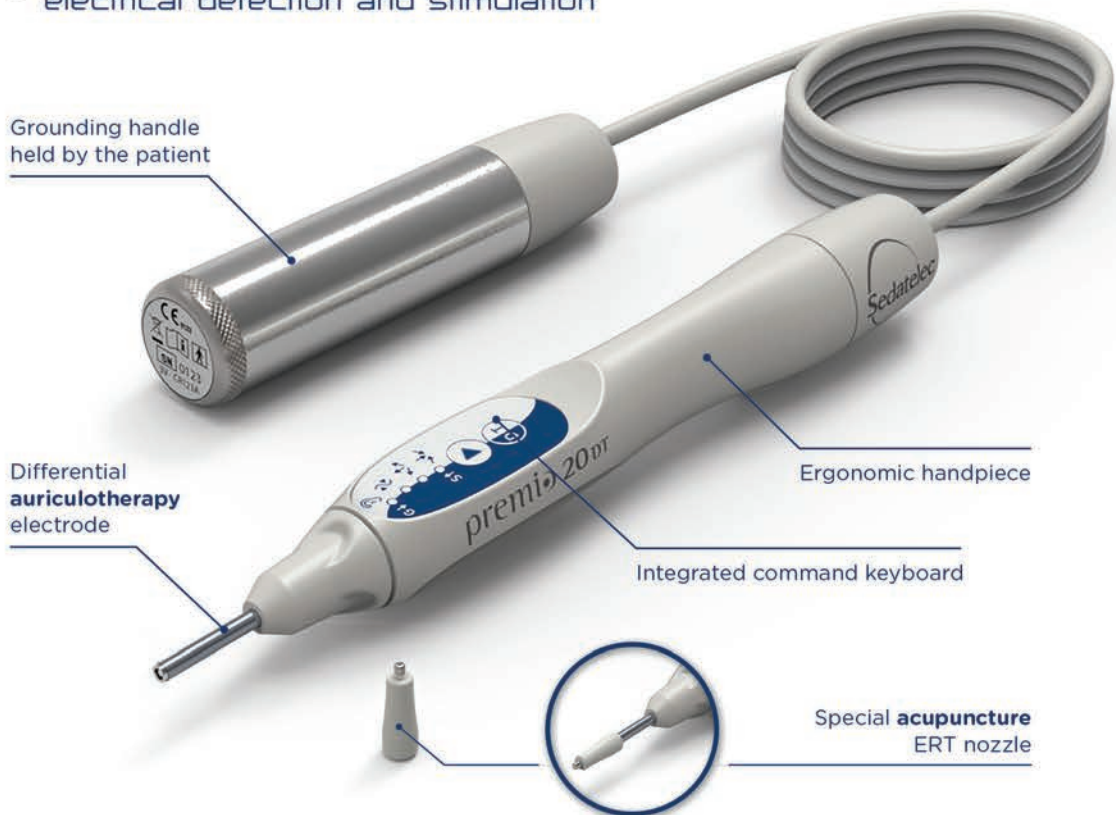

Sedatelec was the original developer of the first auricular point detectors (Diascope®, Servoscope®, Agiscop®) and has concentrated all of its skills and expertise in the production of a new generation, with the **Premio 20**.

Revolutionary design

- PRACTICAL:** no cable or accessory part to install
- ERGONOMIC:** the first device with direct reading from the ear
- ACCURATE:** 9 level differential detection with LED display
- INNOVATIVE:** frequency wobulation in stimulation mode between 1 and 100 Hz
- SIMPLE:** audible trigger threshold adjustment
- COMFORTABLE:** automatic adaptation of stimulation intensity




Detection

- Sufficiently low measurement current not to interfere with the point.
- Constant measurement electrode pressure on the epidermis.
- **Highly accurate** differential detection system (point and environment).
- Low (Gold) or high (Silver) impedance detection possible.
- **Adjustable audible trigger threshold.**
- Continuous measurement display on **9 levels**.
- A grounding handle held by the patient which isolates the practitioner from the measurement circuit.

Differential detection

Measurement of the point environment

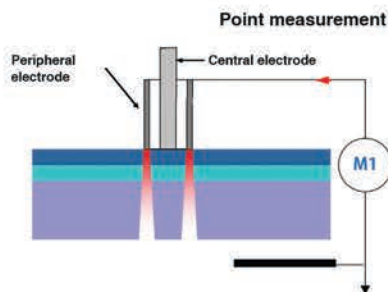

3 levels of led light indication



Slow
flashing



Rapid
Flashing



Fixed
light

Automatic setting of the audible trigger threshold through measurement of the «0» point

D VERSION

DT VERSION





Stimulation

- **Without taking your eyes** off the point detected, simply pressing on the «D/T» button triggers stimulation by scanning the 7 Nogier* frequencies (1.14 Hz included).
- **Automatic adaptation** of stimulation intensity: no pain or burning sensation.
- Emission of a soft and fast «BLEEP» indicating to both patient and therapist that the stimulation is ongoing.
- Duration time set by a louder «BLEEP» every 30 seconds: average time required to incapacitate the point.
- Pressing again on «D/T» restarts detection without taking your eyes off the ear which is being investigated.

*The Nogier frequencies are not recognised by Health Canada.



auriculotherapy




Detection

- **Downward measurement** of the impedance point.
- Special «Body» **ERT nozzle** with soft tip for perfect sliding.
- Progressive diode lighting proportional to the measurement.
- Direct instantaneous reading of the measurement on the stylet confirmed by an «Audible BLEEP».
- **Memorisation of the skin point** detected with the lowest impedance.
- **Reliability:** grounding handle held by the patient isolates the practitioner from the measurement circuit.




Stimulation

- Without taking your eyes off the point detected, simply pressing on the «D/T» button triggers appropriate simulation for the patient's needs.
- **INNOVATION:** emission by frequency wobulation:
 - Alternate for **harmonisation**
 - Decreasing from 100 to 1 Hz in **dispersion**.
 - Rising between 1 and 100 Hz in **tonification**
- Option of working with Nogier* frequency scanning.
- Automatic adaptation of stimulation intensity: no pain or burning sensation.
- Emission of a regular «BLEEP» indicating to the patient and to the therapist that stimulation is ongoing.
- Duration time set by a louder «BLEEP» every 30 seconds.
- Pressing again on «D/T» restarts detection without taking your eyes off the ear which is being investigated.

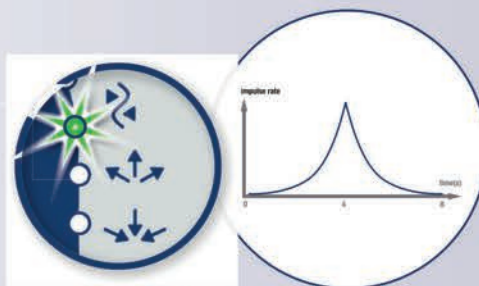


HARMONISATION

This stimulation re-equilibrates the point and circulation in the meridian by alternating accelerations and decelerations of impulse frequency.

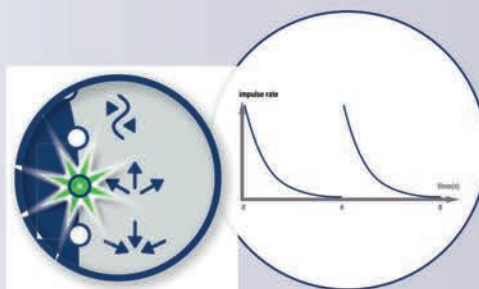

DISPERSION (100 to 1 Hz)

In a few seconds you will disperse the Qi and remove energy stasis. Dispersion is obtained by decreasing the impulse emission frequency.

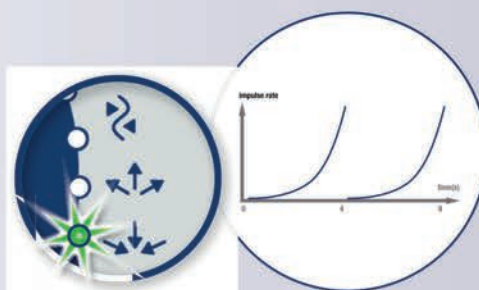

TONIFICATION (1 to 100 Hz)

Rapid controlled stimulation of acupuncture points. Energising effect by accelerating impulse emission frequency.

«BLEEP» every 30 seconds

Effective, light and ergonomic

An ideal
detector/stimulator
for the consulting
room and for
portable use.



Distributor stamp

CE 0123

Manufactured in France



Chemin des Mûriers • F-69540 Irigny (Lyon) • France
Tel: +33 (0) 472 663 322 / Fax: +33 (0) 478 508 903
contact@sedatelec.com • www.sedatelec.com

Before use, read the instructions carefully.

SED-PREMI020-EN-2025-06- PHOTOS ISTOCK / @SEDATELEC